

Wireless Nurse Call Model EPC500WB




FEATURES:

- Surface Mount Emergency Wireless Pull Cord with Inovonics EN1941 Built-In Transmitter
- Red LED and 65db Sounder Upon Activation
 - Battery Powered (Included)
 - Works with Flair ENS24 and ENS32 LED Annunciators
 - Works with other Inovonics compatible alarm panels and call systems

SPECIFICATIONS

MOUNTING	Single Gang Pull Cord mounts to wall or back box with two screws. Use gasket to prevent water from getting inside. Remove for battery replacement.
SIZE	4.5" Tall x 2.8" Wide x 2" Deep
COLOR	White with Emergency Red Pull
POWER	Battery Powered (Included)
INPUTS	Additional NC Dry Contact, Two #22 AWG Wires, for activating Hallway Light or other Device
OTHER	Additional Models with Bed Cord or Ventilator Monitor available

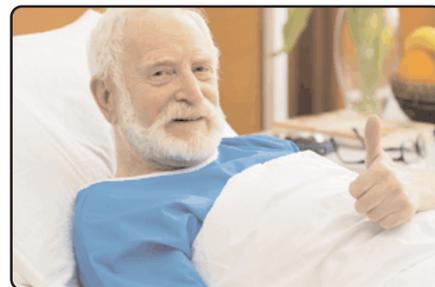

Pull Cord Station

Mount in bathroom or next to bed. Cord and Knob are long enough to reach to floor.

Other Call Devices from Flair:

- Pendants
- Push Buttons
- Motion Sensors
- Magnetic Contacts
- Water Sensors

OPERATION: The Wireless Pull Cord, Model EPC500WB, mounts to the wall with just two screws. A call is initiated by pulling down the red Slide or pulling down red Knob. When a call is initiated in this way the Pull Cord Station sends a wireless signal to an Inovonics receiver. A red LED and Sounder on Pull Cord Station activate upon alarm to notify that a call has been made. Alarm signal restores when Pull Cord slide is reset. The Pull Cord also has an additional NC output for connecting to local alarm light or other device. Use with Flair ENS24, ENS32 and ENS48 LED Alarm Annunciators or other call systems with built-in Inovonics receivers.