


Wireless Nurse Call Model ENS24 and ENS32

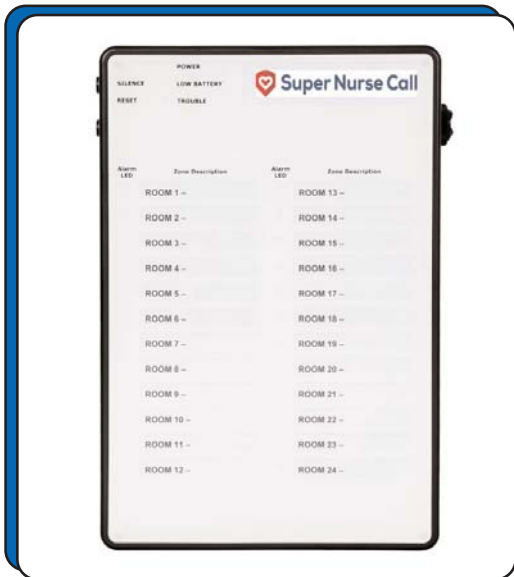

FEATURES:

- Works with Inovonics Transmitters
- Simple, Highly Visual Display with Twenty-Four or Thirty-Two Rooms
- Following or Latching Configuration
 - Removable Clear Cover and Zone Identification Labels
- External Adjustable Alarm Sounder
- One Touch Zone Acknowledge and Reset
- Common Outputs for activating external sirens or other devices

SPECIFICATIONS

MOUNTING	Surface Mount - Enclosure screws to wall with two or four screws. Lid removes with four screws to access control for programming.
SIZE	10" x 15" x 2 1/2"
COLOR	Black Plastic with Clear Zone Identification Cover and Graphic
POWER	Includes Plug-In AC Transformer and Back-Up Battery
INPUTS	Inovonics Wireless Transmitters
OTHER	One Common Open Collector Output and One Form C Relay Output



Nurse Call

Panic Buttons, Pull Cords, Motion Sensors

Accessories from Flair:

- Panic Switches
- Pull Cords
- Motion Sensors
- Magnetic Contacts
- Water Sensors

OPERATION:

Information at a Glance

Flair Wireless Annunciators have a built-in Inovonics serial receiver and work with most Inovonics transmitters. Registration of transmitters is easily performed using the LCD screen and four push buttons on internal control board. Each zone has a red flashing LED for alarm. Alarm can be configured to Follow or Latch on status of transmitters. LEDs at top of annunciator provide status of power, transmitter batteries and transmitter communication. Adjustable alarm sounder is on right side. Top push buttons on left side of enclosure functions as alarm acknowledge and silence. Bottom push button on left side resets alarms when call devices are reset.